# **Dental Conservative – Rubber Dam Instruments**

## **Rubber Dam Stands, Rubber Dam Forceps, Rubber Dam Clamps, Rubber Dam Punch**

**ErgoDenta ApS**

Hørskædden 14, 2630 Taastrup, Denmark.

Tel: +4555241100

Email: info@ergodenta.dk

Web: www.ergodenta.dk

**en-English**

### **Attention**

The operator/user must carefully read and understand this manual thoroughly to keep the product performance durable and reliable for defined life expectancy. After opening the packaging, first of all it is necessary to check the component(s) against the standard configuration. Check that these are all present and in perfect condition.

### **Intended Use:**

Rubber dam instruments are intended for the placement, stabilization, and management of rubber dam systems to achieve isolation of the operative field during dental procedures. They assist in moisture control, improve visibility, and minimize contamination

The Instrument Specific Intended Use is given below:

### **Rubber Dam Stands:**

Intended to be used for holding and organizing rubber dam accessories during procedures.

### **Rubber Dam Forceps:**

Intended to be used for placing and removing rubber dam clamps to secure the rubber dam in position.

### **Rubber Dam Clamps:**

Intended to be used to Stabilize and secure the rubber dam around the tooth to maintain isolation.

### **Rubber Dam Punch:**

Intended to be used to Creates perforations in the rubber dam sheet to accommodate isolated teeth.

 Our Rubber Dam Instruments are intended to be used by qualified dental professionals only in a professional dental facility.

### **Patient population:**

These instruments are intended for use on adult and child patients undergoing dental procedures requiring isolation with a rubber dam. The appropriate instrument type and size must be selected based on the patient's oral anatomy and the specific procedural requirements.

**For Available Configurations/Models please visit the Conservatives section in our Catalogue**

### **Features**

- **Rubber Dam Clamps** are available with serrated and smooth jaws to securely anchor the rubber dam to the tooth, ensuring stability during procedures. These are designed in various shapes and sizes to fit different tooth anatomies.
- **Rubber Dam Stands (Frames)** are made from stainless steel, ergonomically shaped to hold the rubber dam taut and provide unobstructed access to the operative field.
- **Rubber Dam Punch** are designed with precision to create clean and accurate perforations in rubber dam sheets for optimal fit around isolated teeth.
- **Rubber Dam Forceps** feature a locking mechanism and angled tips for secure grip and accurate placement of clamps, even in hard-to-reach areas of the oral cavity.
- **Rubber Dam Pliers** are engineered with precision tips and a spring mechanism for easy placement and removal of clamps, ensuring minimal force is applied to avoid damage to the tooth or clamp.
- All rubber dam instruments are typically crafted from corrosion-resistant stainless steel, ensuring longevity and resistance to repeated sterilization processes.

### **Operating instructions**

 These instruments are non-sterile and must be cleaned and sterilized before first use.

 Always check if the reprocessed instruments are in perfect condition and do not use the instruments if any kind of deterioration, bent or breakage is observed.

 Use a firm but controlled grip to maintain precision and stability. Avoid excessive force that may compromise the integrity of the instrument.

### **For Rubber Dam Stands:**

- Secure the stand in a stable position before placing rubber dam accessories.
- Organize and position accessories for easy accessibility during procedures.

### **For Rubber Dam Forceps:**

- Select the appropriate clamp size and shape based on tooth anatomy.
- Position the clamp securely around the tooth to maintain proper isolation.

### **For Rubber Dam Clamps:**

- Mix dental materials thoroughly before application.
- Ensure consistency and accurate placement when transferring materials.

### **For Rubber Dam Punch:**

- Align the punch to create precise perforations in the rubber dam sheet.
- Ensure the punched holes match the position of the teeth requiring isolation.
- Avoid excessive force when punching to prevent damage to the sheet or misalignment.
- Verify punch sharpness regularly to maintain clean perforations.

 For rubber dam punch and forceps always verify the spring mechanism before use.

### **Adverse Event**

Any serious incident that has occurred concerning the device should be reported to the manufacturer and to the respective competent authority of the Member State in which the incident has occurred.

### **Cleaning & Sterilization**

#### **Cleaning**

 Clean the device with care after each application to avoid the drying of contaminants.

#### **Initial Cleaning**

1. Remove any visible debris from the instrument using moisten a cotton swab or cloth with isopropyl or ethyl alcohol and gently wiping the surface of the product.
2. Use a soft brush for cleaning the instrument if required.

### **Manual Cleaning**

3. Prepare a solution of neutral pH enzymatic detergent according to the manufacturer's instructions.
4. Immerse in a pre-soak enzymatic cleaner solution
5. Thoroughly clean all surfaces of the Instrument using a soft brush or cloth. Pay close attention to any crevices or hard-to-reach areas.
6. Rinse thoroughly under lukewarm running tap water for a minimum of 30 seconds to remove all traces of detergent.

### **Automated Cleaning**

1. Always clean in an appropriate box or cassette and ensure that they are properly positioned within the unit.
2. Always follow the equipment manufacturer's instructions and ensure the instruments are compatible with the cleaning system.
3. Visually inspect for cleanliness, If any contamination is visible, repeat the cleaning steps until required cleanliness is achieved.
4. Dry with clean, lint free cloth or filtered compressed air until there is no visible moisture. Follow STEAM STERILIZATION PROCEDURE.

 Ultrasonic cleaning may be performed in accordance with hospital or facility protocols. Ensure proper rinsing and drying after the process to prevent residue buildup.

### **Sterilization**

Note: Ensure that the instruments are completely dry before sterilization.

### **Steam Autoclaving**

1. Place the instrument in an approved sterilization pouch or wrap.
2. Sterilize using a steam autoclave following a validated sterilization cycle according to ISO 17665-1 and your facility's protocols. A typical recommended cycle is 132-134°C (270-273°F) for 4 minutes.
3. For Rubber Dam Punch and Forceps, ensure that hinges are lubricated with an appropriate medical-grade lubricant after sterilization to maintain smooth operation.
4. proper alignment during sterilization to maintain function
5. Allow the instruments to cool completely before handling. Store in a clean, dry, and controlled environment until ready for use.

6. Follow the pouch/wrap manufacturer instructions for storage conditions and maximum storage time.
7. For Rubber Dam Punch and Forceps periodically check the hinge movement to ensure continued smooth function.

### **Important Notes**

- Always Inspect the instruments for any signs of deterioration, breakage and bent etc. after each sterilization cycle. Discard if damage is observed.
- Always follow your facility's infection control protocols.
- Refer to your autoclave manufacturer's instructions for proper operation and maintenance. The Sterilization equipment must be validated by the hospital and or sterilization equipment manufacturers.
- The above-mentioned cleaning and sterilization guidelines, provided by manufacturer are intended as procedures compatible with specific materials. Sterilization must be performed according to the Hospital/Clinic approved protocol.
- The responsibility for ensuring proper sterilization of instruments lies with the user. The effectiveness of sterilization depends on validated cleaning, packaging, and sterilization procedures carried out at the facility.

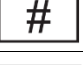
### **Disposal**

Our conservative instruments: Rubber Dam Stands, Rubber Dam Forceps, Rubber Dam Clamps, Rubber Dam Punch, contain no hazardous materials. However, these must be cleaned/sterilized as per defined procedures before disposition. Please follow the local and national regulations or healthcare facilities' defined disposal and waste management policies.

### **Disclaimer**

The products must be used, reprocessed, and maintained strictly in accordance with the instructions provided above. Any deviation from these guidelines by the dental professional or user is undertaken at the user's sole risk. The Manufacturer will not accept any requests for refunds or exchanges under warranty for products that have not been handled and reprocessed in compliance with these instructions.

### **Explanation of utilized symbols**

	Medical Device Compliant with Regulation (EU) 2017/745
	Consult instructions for use
	Medical Device
	Product Code/Catalogue Number
	Model/Variant
	Lot Number
	The Device is supplied non-sterile.
	Sterilizable in a steam sterilizer (autoclave) at 132-134°C (270-273°F)
	Manufacturing Date
	Manufacturer
	Keep in a cool, dry place
	Caution, read instructions

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