
unica master kit

in cooperation with


anterior matrix system


Unica Proximal anterior matrices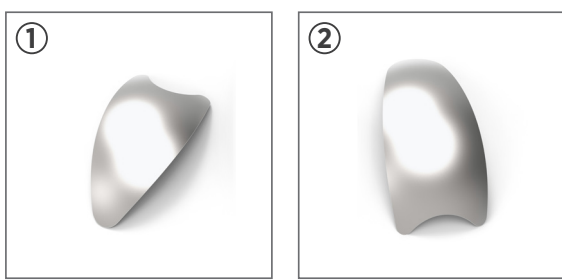


Unica anterior matrices

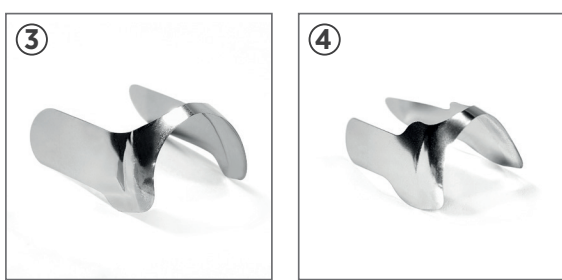How to position Unica proximal matrices

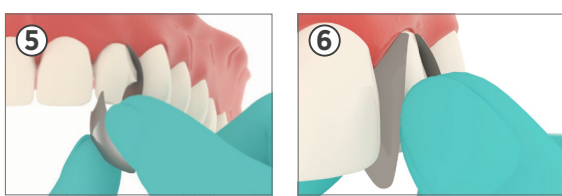How to position Unica matrices

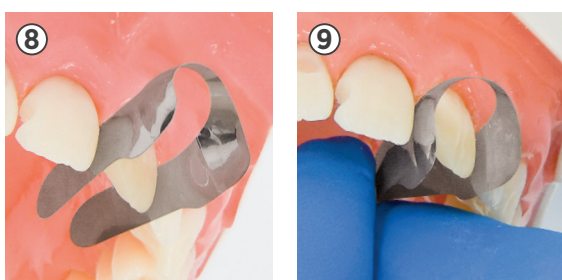Tips and tricks

en - ENGLISH

All individual components are CE marked
Medical Devices Kit for professional use only
Anterior matrix system
Instructions for use
Description
 Unica anterior and Unica minideep anterior, thanks to their contoured shape, adapt correctly to the different morphologies of anterior teeth and are intended to restore interproximal and cervical margins at once, even in presence of rubber-dam or gingival retraction cords reducing chair-time significantly. Furthermore, Unica anterior and Unica minideep anterior matrices, once positioned, allow to easily visualize the final shape of the restoration. Unica anterior & Unica minideep anterior allow to: restore both proximal and cervical margins at once, anatomically restore the proximal margins thanks to its contured profile, easily manage the cervical area. Unica Proximal and Unica Proximal Deep matrices are especially indicated for anterior restorations involving the proximal walls (Class III and IV), peg-shaped lateral incisors, diastema closures, partial and total veneering). The easy-to-place cervical labioapatal arch has the purpose of securing the matrix between the papilla and the tooth subgingivally, in order to improve matrix stability after placement. A wedge can be used to secure the matrix but it is not mandatory. With a shape conceived to ensure an optimal fit on the anterior teeth, it allows better visibility and operatory field access than other solutions such as posterior sectional matrices.
Composition
 Unica anterior & Unica minideep anterior: stainless steel | Unica Proximal & Unica Proximal Deep: stainless steel | myWedge: thermoplastic.
Indications
 Unica master kit is indicated for: Class III cavity restorations, Class IV cavity restorations, Class V cavity restorations, Direct stratification composite veneers, shade modifications.
Contraindications
 None known.
Side effects
 There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydentia.
Interactions with other medical devices or products
 There are no known interactions.
Information for the patient
 No information for the patient related to Unica master kit is necessary.
Preservation and storage
 Store in a clean, dry environment away from moisture or excessive humidity.
Warnings
 Due to the presence of small and sharp components, the use of a rubber dam is mandatory to avoid accidental ingestion or aspiration..

Application
Select an appropriate anterior matrix: Choose ① Unica Proximal or ② Unica Proximal Deep for anterior restorations involving the proximal walls. Use ③ Unica Proximal Deep specifically for periop application with a flat papilla. In demanding cases requiring the restoration of the cervical and proximal areas (complete veneer), use Unica anterior or Unica minideep anterior. ④ Select Unica anterior for: maxillary central incisors maxillary lateral incisors canine teeth. ⑤ Select Unica minideep anterior for: maxillary and mandibular lateral mandibular central incisors canine teeth triangular teeth peg laterals teeth with narrow cervical diameter. **How to position Unica Proximal and Unica Proximal Deep:** ⑤ Select the matrix and gently slide it interproximally. ⑥ Secure the matrix between the papilla and the tooth subgingivally. ⑦ The use of an interproximal wedge (i.e. myWedge) to fix the matrix in position is allowed but not mandatory. **How to position Unica anterior and Unica minideep anterior:** ⑥ Use the placement wings to insert Unica proximal matrices, insert Unica minideep anterior. ⑦ Gently pull the matrix palatally by its placement wings. ⑧ Push the cervical connection into the gingival sulcus. ⑨ Insert myWedge to fix the matrix in position.

Disposal
 Because of their composition, matrices and wedges can not be sterilized and are therefore single use devices. Do not reuse them. Properly dispose devices contaminated with potentially infectious substances of human origin, in accordance with local regulations.

Tips and tricks
 Use Unica anterior and Unica minideep anterior in place with a liquid dam, in case a wedge risks displacing or deforming the matrix.
 ● Yes ● No

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