## Content

English .....	2	Bahasa Malay .....	110
Deutsch .....	8	Malti .....	116
Français .....	14	Nederlands .....	122
Italiano .....	20	Norsk .....	128
Español .....	26	Polski .....	134
Български .....	32	Português .....	140
Česky .....	38	Română .....	146
中文 (中华人民共和国) .....	44	Русский .....	152
Dansk .....	50	Slovenčina .....	160
Eesti .....	56	Slovenščina .....	166
ελληνική .....	62	Srpski .....	172
עברית .....	68	Suomi .....	178
Hrvatski .....	74	Svenska .....	184
íslenska .....	80	ภาษาไทย .....	190
한국어 .....	86	Türkçe .....	198
Latviešu .....	92	Українською мовою .....	204
Lietuvių .....	98	Distributors .....	212
Magyar .....	104		

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Natural bone mineral for bone grafting

### Composition

Geistlich Bio-Oss® is a natural bone mineral of bovine origin. The highly purified osteoconductive mineral structure is produced from natural bone in a multi-stage purification process, adhering to the strictest safety regulations. Geistlich Bio-Oss® is available as a granulate of cancellous bone and as a cancellous-bone block. Because of its natural origin, Geistlich Bio-Oss® is chemically as well as structurally comparable to the mineralised human bone (nanocrystalline natural apatite). Geistlich Bio-Oss® Collagen is a mixture of cancellous-bone granulate and 10% porcine collagen fibres in a block form.

Sterilisation of Geistlich Bio-Oss® and Geistlich Bio-Oss® Collagen is carried out by  $\gamma$ -radiation.

STERILE R

### Properties / Effects

Geistlich Bio-Oss®: The inorganic bone matrix of Geistlich Bio-Oss® has a macro- and micro-porous structure similar to human cancellous bone. Due to the large interconnecting pore volume and the natural composition, the formation and ingrowth of new bone at the implantation site is encouraged. In the course of time Geistlich Bio-Oss® is partially remodeled by osteoclasts and osteoblasts (physiological remodeling). Due to its properties, Geistlich Bio-Oss® is a viable alternative to autologous bone in suitable defects.

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Geistlich Bio-Oss® Collagen: The spongy consistency of the material allows simple trimming. The collagen facilitates adaptation of the Geistlich Bio-Oss® to the defect site. The collagen is slowly resorbed over several weeks.

### Indications

Geistlich Bio-Oss® granulate is recommended for filling of bone defects and bone augmentation, e.g.

- > Augmentation/reconstruction of alveolar ridges
- > Filling of extraction sockets
- > Implantology: preparation of implant sites, filling of bone dehiscences, and sinus floor augmentations
- > Periodontology: filling of bone defects, support of the membrane during guided tissue regeneration (GTR)

Geistlich Bio-Oss® Collagen is recommended for reconstruction of bone defects in maxillofacial surgery and in dental surgery, e.g.:

- > Augmentation/reconstruction of alveolar ridges
- > Filling of extraction sockets
- > Periodontology: filling of bone defects, support of the membrane during guided tissue regeneration (GTR)

Geistlich Bio-Oss® cancellous-bone block is recommended for:

- > Filling of large bone defects

### Instructions for use

The general principles of sterile handling and patient medication must be followed when using Geistlich Bio-Oss® / Geistlich Bio-Oss® Collagen.

- > Complete elimination of granulation tissue following exposure of the defect.
- > Mixing of Geistlich Bio-Oss® granulate with the patient's blood or with physiological saline solution before the implantation.
- > Application:

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- a) Geistlich Bio-Oss® granulate is placed into the defect, using sterile instruments (spatula, spoon or syringe).
  - b) Geistlich Bio-Oss® Collagen may be cut to the required size with sterile tweezers and scissors, in dry form or after moistening (with blood, physiological saline solution). The material is applied using sterile instruments, e.g. tweezers.
  - c) Geistlich Bio-Oss® cancellous-bone blocks may be adapted to the desired size with a scalpel and placed into the bone cavity. If necessary, the block is moistened with the patient's blood or with physiological saline solution.
- > In situ modeling may be performed with a sterile spatula or other suitable instrument.
  - > It is advisable to cover Geistlich Bio-Oss® with a membrane barrier (e.g. Geistlich Bio-Gide®).
  - > When closing the wound, the soft tissue flap should completely cover the implanted Geistlich Bio-Oss®, and should be fixed by sutures.
- > If primary wound closure cannot be achieved completely, further mobilisation of the flap (incision through the periosteum) should be performed or wound covering should be achieved with a membrane (e.g. Geistlich Bio-Gide®).

### Special instructions for use in periodontology

- > A basic requirement for successful periodontal treatment includes control of any bacterial infection as well as thorough oral hygiene. It is therefore advised that, preceding the surgical intervention, there be a hygiene phase, which would include proper instruction for the patient. A postoperative maintenance phase can ensure long-term therapeutic success.
- > Besides plaque control, the filling of periodontal defects with Geistlich Bio-Oss® or Geistlich Bio-Oss® Collagen requires successful local treatment of the periodontal lesion (root planing, debridement), prior to the implantation. The defect should be covered

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with a membrane (e.g. Geistlich Bio-Gide®) for optimal tissue regeneration.

### Limitation on use / Precautions

#### Contraindications

- > Geistlich Bio-Oss® / Geistlich Bio-Oss® Collagen should not be used in the presence of infected wounds.
- > Geistlich Bio-Oss® Collagen should not be used in patients with a known allergy to collagen.

#### Precautions

Geistlich Bio-Oss® and Geistlich Bio-Oss® Collagen should only be used by trained dentists and surgeons.

Geistlich Bio-Oss® / Geistlich Bio-Oss® Collagen should be used with special caution in patients with

- > acute or chronic infection (e.g. osteomyelitis) at the surgical site;

- > uncontrolled metabolic disorders e.g. diabetes, osteomalacia, thyroid disorder, severe renal or hepatic disorder
- > prolonged corticosteroid therapy
- > autoimmune disease
- > radiotherapy
- > heavy smoking

In order to ensure the regeneration of bone, Geistlich Bio-Oss® should only be implanted in vital bony tissue and in direct contact with the host bone (if necessary with microfracture of the bone surface). For larger defects, the addition of autologous cancellous bone or bone marrow may improve the regeneration process. From experience, mobility by mechanical loading (compression loading) or insertion of implants (two-stage procedure) in the augmented area should be avoided until several weeks after the insertion of Geistlich Bio-Oss® / Geistlich Bio-Oss® Collagen. From experience, mechanical loading (compression loading) of Geistlich Bio-Oss®/Geistlich Bio-Oss® Collagen augmented areas is possible after 6 months at the

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earliest. According to this, the appropriate timing of implant insertion depends on the residual local bone volume.

### Side effects

Incompatibility reactions with Geistlich Bio-Oss® / Geistlich Bio-Oss® Collagen cannot be totally excluded. As Geistlich Bio-Oss® Collagen contains collagen, in very rare cases allergic reactions may occur. Possible complications which may occur with any surgery include swelling at the surgical site, flap sloughing, bleeding, local inflammation, bone loss, infection, or pain.

### Pregnancy / Lactation

Data on the use of the product during pregnancy or lactation are not available. For safety reasons: women who are pregnant or breastfeeding should not be treated with Geistlich Bio-Oss® / Geistlich Bio-Oss® Collagen. The safety and efficacy of Geistlich Bio-Oss® / Geistlich Bio-Oss® Collagen have not been investigated in children before skeletal maturity.

6

### Other Information

The content of the bottle or the blister-pack is intended for single use.

Re-sterilisation is not possible. Discard any unused material. Sterility and function are no longer guaranteed in the event of reuse.

Do not use the product after the indicated expiry date.

Store in a dry place and at room temperature (15–25° C).

The product is sterile, if the packaging is unopened and undamaged. If the sterile packaging has been damaged, the product must not be used.

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Vials of 0.25 g, 0.5 g and 2.0 g

Geistlich Bio-Oss® Collagen:

Double-blister pack with 1 preformed block of cancellous-bone granulate and collagen.

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**Bahrain**

Gulf Pharmacy & General  
Store  
a division of Gulf  
Corporation of Technology  
P.O. Box 2576, Villa 390  
New Zinj 330  
Manama  
Kingdom of Bahrain

Yousuf Mahmood Husain  
W.L.L.  
P.O. BOX 23  
Bldg. 97, Block 328  
Road 322  
Salmaniya Ave.  
Manama  
Kingdom of Bahrain

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**Estonia**

Sirowa Tallinn AS  
 Salve 2C  
 111612 Tallinn  
 Estonia

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**Hong Kong**

HEAT Consultancy Co. Ltd.  
 1912-13, The Centre  
 99 Queen's Road Central  
 Hong Kong

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E. Bridde ehf  
 Holtasmári 1  
 201 Kópavogur  
 Iceland

**India**

LifeCare Devices Private  
 Limited  
 LifeCare House  
 841, Mori Road  
 Mahim West  
 Mumbai – 400016  
 India

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Dent-Med Materials B.V.  
Dorpsstraat 72  
1713 HK Obdam  
Nederland

**New Zealand**

Henry Schein Shalfoon  
23 William Pickering Drive  
Albany, Auckland 0632  
New Zealand

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**Philippines**

Spectrumed Inc.  
 Unit 2001-B West Tower  
 Philippine Stock Exchange  
 Center  
 Exchange Road, Ortigas  
 Center  
 Pasig City 1605  
 Philippines

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**Singapore**

Raydent Supplies (S) Pte Ltd  
6 Ubi Road 1 #04-01  
Wintech Centre  
Singapore 408726  
Singapore

**Slovakia**

Sanitaria spol. s.r.o.  
Sabinovská 4/A  
821 02 Bratislava  
Slovenská

**Slovenia**

Aurora Dental d.o.o.  
Brilejeva Ulica 2  
1000 Ljubljana  
Slovenija

**South Africa**

Implants for Africa (Pty) Ltd.  
Unit 30B  
Platinum Junction Business  
Park  
School Street  
P.O. Box 1028  
ZA-7441 Milnerton

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**Vietnam**

D.O.E.  
 (Dentistry Of Excellence)  
 199 Nguyen Thi Minh Khai  
 Dist.1 – Ho Chi Minh City  
 Vietnam

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