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| EN | Reusable dental tools for handling and application of prepared measure of MTA+ material |
|----|-----------------------------------------------------------------------------------------|
|----|-----------------------------------------------------------------------------------------|

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**

NON - STERILE PRODUCT   STERILIZE   DIRECTLY BEFORE USE

The tools are made of   corrosive – resistant stainless steel.

**COMPONENTS**  
Product contains:

- HANDLE
- PISTON

The piston is used to squeeze the cement to the treatment area.

**HANDLING DENTAL TOOLS MADE OF STAINLESS STEEL**  
Special processes have been undertaken during the manufacture of the dental instruments to ensure corrosion-resistance. In spite of this, the durability of these dental instruments depends on their proper use and correct maintenance.

**CLEANING, RINSING AND WASHING**  
The packaged instruments are non – sterile. Before sterilization, these new instruments should be washed thoroughly with warm water and using detergent. Before each use these dental instruments must be sterilized.

**STERILIZATION**  
All (reusable) dental instruments must be disinfected, then washed out to remove detergent and then dried before sterilization. Ultrasonic cleansing is recommended in particular as the most effective sterilization. Instruments should be sterilized in accordance with national requirements.

Recommended sterilisation methods:

| Method          | Minimal parameters:  |
|-----------------|----------------------|
| Steam autoclave | 134°C, for 5 minutes |

**PRECAUTIONS**  
Adhere to the guidelines for cleaning and sterilization of the dental instruments.  
If a serious product-related event develops, report it to the manufacturer and the competent authority of the state in which the user or patient is domiciled.

**STORAGE**  
Dental instruments should be stored in temperatures ranging between 5-30°C and relative humidity below 70%. Before use protect the instrument against contamination or damage of the material structure (cracks, bends, fractures, peeling, etc.). Failure to adhere to the above stated instructions decreases instrument durability and reduces instrument lifetime.

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