
VOCO

Caries Marker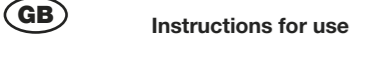

CE ⁰⁴⁸²

Instructions for use

Product description:

Caries Marker contains a colour indicator (acid red) to mark infected, carious dentine. The **Caries Marker** thus enables the visual control of carious areas. Healthy as well as uninfected, remineralisable dentine does not turn red, enabling gentle excavation. If possible, uninfected (not dyed) dentine should be preserved.

- Indications:**
- Caries Marker** dyes carious dental substance:
 - to reveal remaining carious areas after removal of the visible caries
 - to provide a clear boundary between carious and healthy dentine, especially on secondary dentine which is different in shade
 - to check if caries has been removed completely
 - to trace carious areas which are difficult to see
 - to demonstrate carious areas to the patient
 - for training and teaching purposes (on extracted teeth)
 - for finding obliterated root-canals
 - for finding micro-cracks in fillings and margins

Application:

First remove the softened and clearly discoloured dentine. Wet the cavity with **Caries Marker**, allow it to act for 5 - 10 s (**not longer!**) and then rinse away with water spray. For best results, apply **Caries Marker** with **Pele Tim** foam pellets. Carious dentine will turn bright red. Remove dyed dentine areas completely, taking care to conserve the undyed dentine areas. Repeat the application until no further dentine colouration occurs. Plaque and residual caries in enamel will equally be dyed red by **Caries Marker** and should also be removed.

Advice and precautions:

The intensity of dyeing can be increased by slightly drying the cavity. Application should not exceed 10 s. Remove all dye residue by rinsing with water. Otherwise there is the risk of dyeing and removing dentine which is not carious. Do not use in direct proximity of the pulp, to avoid the risk of confusing **Caries Marker** with blood. Avoid contact with skin and clothes (to prevent discolouration). Clean contaminated clothes with diluted vinegar (after verifying colour stability). **Caries Marker** contains propylene glycol. Hypersensitive patients may have an allergic reaction or experience irritation if undiluted **Caries Marker** comes in contact with the skin or oral mucosa. **Caries Marker** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.

Storage:

Store at temperatures between 4 °C - 23 °C. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and /or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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