PoloDent Cold Spray is meant for quick freezing of foam pellets which are then used for vitality testing of teeth in dentistry as well as for cooling impression materials such as hydrocolloid etc. For professional users only!

Notes on application: Spray the object to be cooled from a distance of approx. 2-3cm for 1-2 seconds. Do not use directly in the mouth, danger of freezing! Use only outside or in well-ventilated rooms.

Safety instructions: H 222 - H 229 Extremely flammable aerosol. Pressurised container: May burst if heated. P2210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P251 Do not pierce or burn, even after use. P410+P412 Protect from sunlight and do not expose to temperatures exceeding 50°C/122°F. P211 Do not spray on an open flame or other ignition source. Keep out of reach of children. Contains orange terpenes: May cause allergic reactions.

Notes on incidents: All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority of the Member State in which the user/patient is established.

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