FIG

# Honigum Pro

## Putty

## Inhalt | Contents

Gebrauchsinformation – DEUTSCH	4
Instructions for use – ENGLISH	13
Mode d'emploi – FRANÇAIS	21
Istruzioni per l'uso – ITALIANO	31
Instrucciones de uso – ESPAÑOL	40
Instruções de utilização – PORTUGUÊS	50
Gebruiksaanwijzing – NEDERLANDS	59
Brugsanvisning – DANSK	68
Bruksanvisning – SVENSKA	76
Instrukcja użycia – POLSKI	85
Инструкция по применению – РУССКИЙ	95

## Produktbeschreibung

---

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---

Vinylpolysiloxane, Siliciumdioxid, Pigmente, Additive, Platinkatalysator

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## Product description

---

Honigum Pro/Putty is an impression material based on addition curing silicones.

The material is available as a hand-mix version and in the ready-to-use pre-filled MixStar cartridge, which is suitable for use in the DMG MixStar-eMotion and other commercially available mixing machines.

## Indications

---

- Crown and bridge impressions
- Inlay and onlay impressions

## Basic safety instructions

---

- For dental use only!
- Keep away from children!

## Advice on application


---

- The working times (refer to “Technical data” section on page 19) apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.
- Rinse off residues of retraction solutions thoroughly with water because they can increase the setting time in some instances.
- Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- Never allow the impression material to come into contact with latex gloves. DMG recommends the use of PVC or PE based gloves.
- Do not mix Honigum Pro/Putty with condensation curing silicone or polyether impression materials.


## Using the MixStar cartridge in the DMG MixStar-eMotion

---


- ▶ Follow the instructions for use for the „MixStar-eMotion“!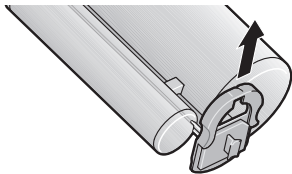


1. Slide the locking mechanism upwards in order to open the cartridge. Do not remove the locking mechanism from the cartridge.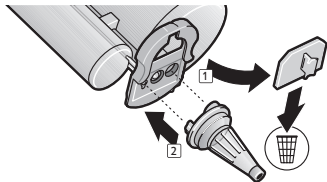

2. Remove the cap from the cartridge and discard. Attach a new mixing tip.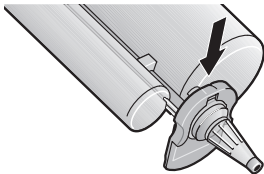

3. Secure the mixing tip by sliding the locking mechanism into the locked position.

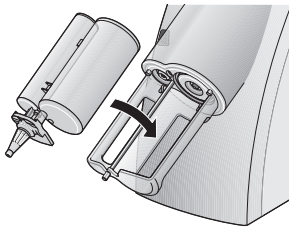4. Insert the cartridge into the MixStar-eMotion. A brief acoustic signal confirms the correct insertion of the cartridge.

**Note:** Prepare fresh cartridges for use by extruding and discarding the first 3 cm of impression material.

Leave the used mixing tip on the cartridge after use. It serves as a cap.

## Using the MixStar cartridge in mixing units from other manufacturers

---

- ▶ Follow the instructions of the mixing unit you are using!
- ▶ First insert the cartridge into the mixing unit without the mixing tip. Otherwise appropriate mixing of the material cannot be guaranteed!
- ▶ In this case proceed as described in the section “Changing the mixing tip” section on page 15

## Changing the mixing tip

---

- ▶ Follow the instructions of the mixing unit you are using!

**Note:** Attach a new mixing tip before each mixing process.

1. Slide the locking mechanism upwards.

2. Remove the old mixing tip and any hardened material blocking the cartridge openings.
3. Place the mixing tip securely on the respective cartridge opening.
4. Lock the mixing tip in place with the locking mechanism.

## Using the hand-mix system

---

- ▶ For the individual times refer to the “Technical data” section on page 19.
1. Take equal amounts of the base paste and the catalyst paste out of the container, always using the colored spoon for the base paste and the white spoon for the catalyst paste.
  2. **Mixing:** At room temperature, knead the base and catalyst pastes in a ratio of 1:1 with the finger tips for 30 seconds, until the color is homogenous.
- ▶ Reseal the container immediately after use! Do not mix up the container lids!

## Recommended use

---

- Honigum Pro/Putty can be used in all common impression trays.
- In order to improve the adhesion of the material to metal or plastic trays, a tray adhesive for A-silicon (e.g. Tray Adhesive, DMG) should be used.

## Double-mix impressions

- ▶ For the individual times applicable to the materials used refer to the “Technical data” section. Observe the associated instructions for use!
- ▶ Fill the tray with Honigum Pro/Putty.

**Information for MixStar cartridges:** To avoid air bubbles, the end of the mixing tip should stay immersed in the impression material throughout the process.

- ▶ Syringe the preparation with Honigum Pro/Light.

**Note:** The filling of the tray and syringing should be completed at the same time as the higher temperature in the mouth leads to a quicker setting of the syringed material compared to the tray material.

- ▶ Position the filled tray in the mouth within the working time, slowly and using very little force, and allow the impression to form.
- ▶ Remove the impression from the mouth once the setting time has elapsed, refer to “Removal” section on page 17.

### **Corrective impressions**

- ▶ For the individual times applicable to the materials used refer to the “Technical data” section. Observe the associated instructions for use!

**Note:** No more than 30 minutes are permitted to pass between the pre-impression and correction impression, otherwise it is not possible to guarantee a firm bond between the two impression materials. Cleaning with alcohol is also not

permitted between pre-impression and correction impression.

- ▶ Cut generous drainage channels into the pre-impression and remove imperfections.
- ▶ After experimental repositioning, thoroughly rinse the impression with water and dry.
- ▶ Apply Honigum Pro/Light around the preparation and on the pre-impression.
- ▶ Position the filled tray in the mouth within the working time, slowly and using very little force, and exert brief initial pressure.
- ▶ Remove the impression from the mouth once the setting time has ended, refer to “Removal” section on page 17.

### **Removal**

---

- ▶ Remove the impression parallel to the longitudinal axis of the prepared teeth.

- ▶ Then proceed as described under “Cleaning and disinfecting the final impression” section on page 18.

## **Cleaning and disinfecting the final impression**

---

- ▶ Only disinfect the final impression with dipping solutions. DMG recommends the use of a 2% glutaraldehyde solution.
- ▶ Please ensure that the disinfectant used is indicated for use with addition curing silicone impression material.
- ▶ Follow the instructions of the disinfectant solution’s manufacturer precisely!
- ▶ Leave the final impression to dry and store until casting, refer to “Storage of the impression” section on page 18.
- ▶ For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0; [www.dgzmk.de](http://www.dgzmk.de)

- ▶ ADA recommends following Council on Dental Materials, Instruments and Equipment. Disinfection of impressions. JADA 1991;122(3):110; [www.ada.org](http://www.ada.org)

## **Casting and galvanization**

---

- ▶ The impression should be cast 30 minutes at the earliest after removal from the mouth.

DMG recommends using special purpose hard plaster (ISO 6873, Type 3; e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for the forms (ISO 6873, Type 4; e.g. Fujirock II, GC). The impression can be copper or silver-plated.

## **Storage of the impression**

---

- ▶ Store the impression in a dry place at room temperature (15–25 °C/ 59–77 °F)!
- ▶ Protect against exposure to direct sunlight!

The impression is stable and can be cast even after months without loss of accuracy.

## Technical data

<b>Honigum Pro/Putty</b>		 Soft
Mixing time [s]	N. A.	≈ 30
Working time* [min]	≤ 1:45	≤ 3:15
Time in the mouth [min]	≥ 3:15	≥ 3:15
Dimensional change after 24h [%]	≤ 0.5	≤ 0.5
Compression set [%]	≤ 0.6	≤ 0.5
Strain in compression [%]	≈ 2.0	≈ 2.2

<b>Honigum Pro/Putty</b>	 Soft Fast	 Rigid Fast
Mixing time [s]	≈ 30	≈ 30
Working time* [min]	≤ 1:45	≤ 1:45
Time in the mouth [min]	≥ 1:45	≥ 1:45
Dimensional change after 24h [%]	≤ 0.5	≤ 0.5
Compression set [%]	≤ 0.5	≤ 0.5
Strain in compression [%]	≈ 2.2	≈ 1.7

\*The times apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.  
The working time describes the period from the start of mixing until the time by which the tray must be positioned in the patient's mouth.

## Composition

---

Vinyl polysiloxanes, silicium dioxide, pigments, additives, platinum catalyst

## Classification

---

ISO 4823:2000, type 0: Putty consistency

## Storage and shelf life

---

- ▶ Store in a dry place at room temperature (15–25 °C/ 59–77 °F)!
- ▶ Do not store in the refrigerator or freezer!
- ▶ Protect against exposure to direct sunlight!
- ▶ Leave the used mixing tip on the cartridge after use. It serves as a cap.
- ▶ Reseal the jar immediately after use!
- ▶ Do not use after the expiration date!

## Available in the following forms

---

### Not available in the USA

1 cartridge with 380 ml paste, 10 MixStar-Tips	
Honigum Pro/Putty	
	REF 929374

5 cartridges with 380 ml paste, 50 MixStar-Tips	
Honigum Pro/Putty	
	REF 929375

2 jars with 450 ml base and catalyst paste	
Honigum Pro/Putty	
 Soft	REF 929364
 Soft Fast	REF 929365
 Rigid Fast	REF 929366

8 jars with 450 ml base and catalyst paste		
Honigum Pro/Putty		
	Soft	REF 929369
	Soft Fast	REF 929370
	Rigid Fast	REF 929371

### Only available in the USA

1 cartridge with 380 ml paste, 10 MixStar-Tips		
Honigum Pro/Putty		
		REF 989374

**Caution:** US Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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**DMG** Chemisch-Pharmazeutische Fabrik GmbH