

Indirect Pulp Capping Placement

Dentistry courtesy of Dr. Ross Nash



1 Indirect pulp capping. Infected dentin is removed and affected dentin remains allowing for incremental layers of TheraCal LC to be placed.



2 TheraCal LC directly applied to preparation in 1mm incremental layers and manipulated into a smooth surface.



3 Each increment is light-cured for 20 seconds.



4 Surrounding enamel is etched using BISCO's SELECT HV® ETCH.



5 BISCO's ALL-BOND SE® is applied and light-cured.



6 Final prep after core build-up using BISCO's CORE-FLO™ DC.

Ordering Information

4 SYRINGE PACKAGE

4 Syringes TheraCal LC (1g ea.), Accessories, Instructions

H-33014P

REFILLS

1 Syringe TheraCal LC (1g), Accessories, Instructions

H-3301P

50 Black Disposable Syringe Tips (22 Gauge)

X-80621N

TO ORDER CALL:

800-247-3368 • 847-534-6000

www.bisco.com



MC-2334TC

The
**SECURITY
BLANKET**
for
**Pulpal
Protection**

TheraCal LC®



**Seal and Protect with TheraCal LC
Pulp Capping Material and Liner**



Rx Only

