DIRECTIONS FOR USE

see insafedental.eu for demo video

YOUNG®
INNOVATIONS

Index

EN	Instructions for use	3
DE	Gebrauchsanweisung	12
ES	Instrucciones de uso	21
FR	Notice D'utilisation	30
IT	Modalità D'impiego	39
NL	Gebruiksaanwijzing	48
PL	Instrukcja Użytowania	57
SV	Bruksanvisning	66

Instructions for use

1

Sharps Container Fitting

[1a]


[1b]

Position base on a smooth clean worktop and push down firmly to secure.

[1c]

Slide sharps container into base until it clicks into place.

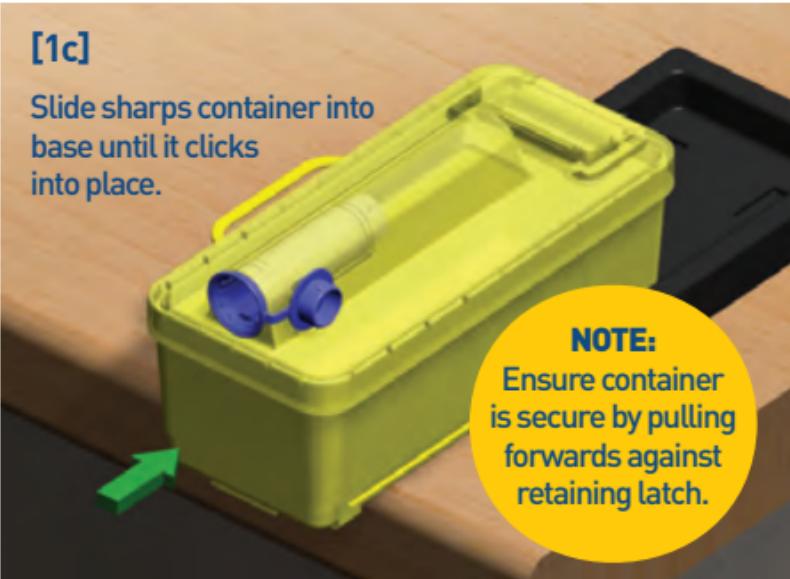**NOTE:**

Ensure container is secure by pulling forwards against retaining latch.

2

Needle Loading

[2a]

Align hole on needle adaptor with slot on body and push fully home.

[2b]

Remove rear needle cover and screw needle fully on.

[2c]

Turn sleeve collar fully clockwise (arrows misaligned).

[2d]

Slide sleeve forward to fully extended position.

[2e]

Turn sleeve collar fully anti-clockwise to lock (arrows aligned).

**WARNING:**

Ensure that the sleeve lock collar is fully activated. If it is not then accidental needle stick injury may occur.

3

Cartridge Loading

[3a]

Pull plunger back against spring and hold.

**[3b]** Slide cartridge into syringe body opening and release plunger to secure.**4**

Syringe Use

- Grip sleeve lock collar and turn fully clockwise (arrows misaligned) **[2c]**.
- Slide sleeve fully back.
- Turn lock collar anti-clockwise to lock (arrows aligned) **[2e]**.
- Give injection as normal.
- Grip sleeve lock collar and turn fully clockwise (arrows misaligned) **[2c]**.
- Slide sleeve fully forward **[2d]**.
- Turn lock collar anti-clockwise to lock (arrows aligned) **[2e]**.
- Repeat above as required for further injections.
- Load new cartridge as shown in section 3.

5

Changing Cartridge

- With sleeve in forward locked position hold plunger stem **[3a]**, pull back against spring and hold.
- Grip cartridge with index finger and thumb through upper and lower syringe body apertures.
- Push cartridge upward and back to remove.
- Load new cartridge as shown in section 3.

6

Needle Removal


[6a] With sleeve locked in the forward position, enter the syringe into the sharps container lid aperture.

[6b] Holding syringe 'T' handle, and exerting slight forward pressure, turn the syringe anti-clockwise a small amount to unlock, and push syringe forward into container.

[6c] Push syringe fully forward - syringe is now 'locked' into container.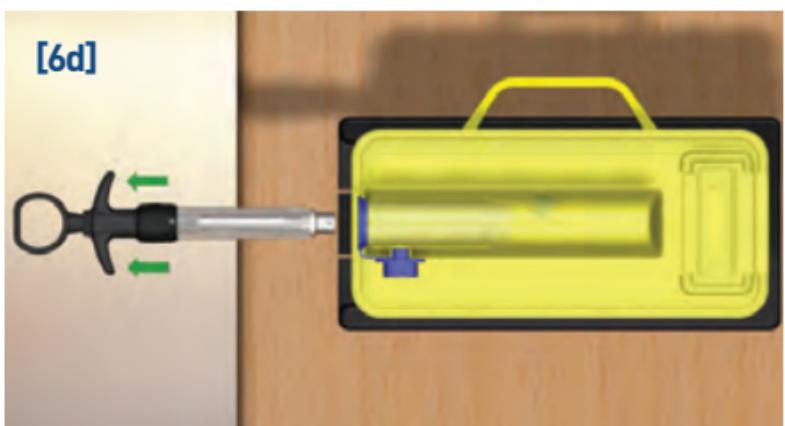

[6d] With index and second fingers under 'T' handle, pull syringe sharply backwards to remove the needle.

7 Sharps Container Removal

- When the container has been filled to the maximum indicator line disposal is required.
- Close the forward and rear container apertures and push fully home to lock.
- Press down on the lower latch feature on the base and pull the container forward and away from the base.
- Holding the container by the attached handle remove it to a safe area for disposal collection.
- Load an empty container onto the base as shown in section 1.

8

Cleaning and Sterilisation

The syringe is supplied non-sterile, and should be autoclaved before first use.

Cleaning

- Wash the surface of the syringe to remove debris, blood and saliva that may be present.
- The syringe can be cleaned in an ultrasonic bath using the manufacturer's recommended detergent.

Disinfection

- The syringe should be disinfected using a registered hospital use surface disinfectant.
- Apply the disinfectant by wiping onto the external surfaces of the syringe using a suitable towel or cloth. Do not spray directly onto the syringe surfaces. Do not disinfect the syringe by immersion.

Sterilisation

- Following cleaning and disinfection, as above, unscrew the plunger retaining nut and separate the plunger assembly. With the syringe sleeve locked in the forward position, place both syringe parts in a suitable autoclave pouch.
- Place in an autoclave and sterilise using a standard steam autoclave cycle to 134°C for a minimum of 3 minutes.
- Use only steam autoclave sterilisation method.

PLEASE NOTE:

Do not remove and load a second needle onto the same syringe during a patient procedure. If a new needle is required then use a clean sterilised syringe.

9

Changing Outer Sleeve

[9a]

Unlock sleeve and move to position shown.

[9b]

Insert suitable tool into hole on collar, push down and separate collar from sleeve.

[9c]

Remove small locking piece and slide sleeve off syringe body.

[9d]

Replace sleeve with new part, insert locking piece, push tab inward and slide collar into place.

[9e]

Check operation of lock collar and sliding action.

10

Syringe Types**Standard Syringe**


- > Syringe option with 'T' plunger handle and plain plunger end is non-aspirating.

**Aspirating Syringe****Cartridge Seal Type**

- > With the syringe needle and cartridge loaded as above push lightly on the plunger to engage the plunger end into the cartridge bore.

- > Prior to giving the injection expel a small amount of anaesthetic.
- > Enter the needle into the tissue of the desired operation site.
- > When the required needle penetration depth is reached draw back the syringe plunger slightly (2mm) and check for signs of blood in the forward (needle distal end) area of the cartridge.
- > If blood is evident withdraw the needle and re-enter the tissue at a different site. Repeat the above until no blood is seen.
- > At that point administer the amount of anaesthetic required.

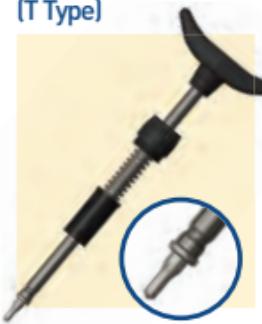Self Aspirating Cartridge Type**Self Aspirating (T Type)****Self Aspirating (Thumb Ring)**

For use with self-aspirating cartridges

Harpoon Type**Harpoon Aspirating (Thumb Ring)**

IMPORTANT NOTICE - PLEASE READ

InSafe local anaesthetic Syringe System is intended for use solely by appropriately qualified or trained dentistry professionals and staff and is designed for use solely with InSafe disposable needle adapters.

To avoid physical harm to patients, or damage to equipment and/or malfunction, use only adapter and other syringe components made by Young Innovations for the InSafe Syringe System, and always follow the manufacturer's instructions and recommendations. Failure to do so will be entirely at the user's own risk and Young Innovations cannot be responsible for any loss or damage arising in any way as a result of a failure to follow such instructions and recommendations.

Any express or implied warranty is excluded to the extent permitted at law.

Report any serious injury in relation to this device to the manufacturer and the competent authority of the Member State in which the incident occurred.

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Astek Innovations Ltd.
Young Microbrush Ireland Ltd
MDSS CH GmbH