# HySolate SyntX Dam

Non-Latex Pre-Printed  
Polyisoprene Dental Dam

Instructions for Use

# Table of Contents

<b>EN</b> Instructions for Use .....	3
<b>DE</b> Gebrauchsanweisung.....	9
<b>FR</b> Mode d'emploi.....	16
<b>ES</b> Instrucciones de uso .....	23
<b>BG</b> Инструкции за употреба .....	30
<b>CS</b> Návod k použití.....	37
<b>DA</b> Brugsanvisning.....	44
<b>EL</b> Οδηγίες χρήσης.....	51
<b>ET</b> Kasutusjuhend .....	58
<b>HR</b> Upute za upotrebu .....	65
<b>HU</b> Használati útmutató .....	72
<b>IT</b> Istruzioni per l'uso .....	79
<b>LT</b> Naudojimo instrukcijos.....	86
<b>LV</b> Lietošanas instrukcija .....	93
<b>MT</b> Istruzzjonijiet għall-Użu .....	100
<b>NL</b> Gebruiksaanwijzing.....	107
<b>PL</b> Instrukcja użycia.....	114
<b>PT</b> Instruções de utilização.....	121
<b>RO</b> Instrucțiuni de utilizare .....	128
<b>SK</b> Návod na použitie.....	135
<b>SL</b> Navodila za uporabo .....	142
<b>SV</b> Bruksanvisning .....	149
<b>TR</b> Kullanma Talimatları.....	156

**Intended Use/ Indications for Use:**

Dental Dam is intended to be used intraorally to isolate one or more teeth and create a barrier to the surrounding area during a dental procedure.

**Contraindications:**

There are no contraindications.

**Intended Users:**

- Dental Practitioners
- Dental Assistants

**Patient Target Group(s):**

Suitable for all patient groups.

**Clinical Benefits:**

The use of Dental Dam has the following clinical benefits:

- Increased visibility through tooth/teeth isolation
- Reduced risk of the patient aspirating or swallowing instruments, tooth debris, medicaments or irrigation solutions
- Creation of a dry operating field
- Protection of soft tissue from contact with instruments
- Infection control barrier

**Characteristics:**

- Two sizes: 5" x 5" or 6" x 6" (127 mm x 127 mm or 152 mm x 152 mm)
- Non-Latex

**Storage:**

Store Dental Dam between 50°F - 104°F (10°C - 40°C).

Keep dry and keep away from heat.

### **Residual Risks, Warnings and Precautions:**

Refer to the Warnings and Precautions section for disclosure of risk associated with the use of the Dental Dam in line with its Intended Use.

### **Instructions**

**Note:** If intending to use non-latex dental dam, always ensure the correct dam is chosen prior to use. To prevent mix-ups or cross contamination do not store near latex dental dam.

Dental Dam is intended to be used with various other devices including clamps and frames.

Refer to the instructions for use for the other devices being used.

The following general steps are provided as guidance and can be tailored to fit the specific case:

**Step 1:** Determine the Operating Field.

**Step 2:** Dental Dam Preparation.

- Determine hole positions
- Mark hole positions
- Punch holes

**Step 3:** Clamp/Retainer Selection

**Step 4:** Clamp/Retainer Application

**Step 5:** Dental Dam Placement

## Parts List

**Note:** Not all Catalog Numbers are available in all regions

Catalog #	Description
60037566	HySolate SyntX Dam Non-Latex Pre-Printed Blue 5" x 5" (127 x 127 mm), 30 pcs
60037567	HySolate SyntX Dam Non-Latex Pre-Printed Blue 6" x 6" (152 x 152 mm), 30 pcs
60037568	HySolate SyntX Dam Non-Latex Pre-Printed Blue 6" x 6" (152 x 152 mm), 75 pcs

## Warnings and Precautions

Warnings	Statement
	Failure to properly follow the directions in this IFU may result in patient injury or compromise the integrity of the dental dam. Use the product only in accordance with its intended use, see Intended Use.
	Do not reuse Dental Dam. Dental Dam is intended to be used only once. Reuse of Dental Dam intraorally is a biological hazard and can result in cross contamination.
	Dental Dam should be inspected prior to use. Discard torn or brittle Dental Dam. Use of Dental Dam that has lost its performance characteristics can result in loss of functionality and compromise the integrity of the procedure.

Precautions	Statement
	Do not expose Dental Dam to high temperatures. Store Dental Dam away from heat sources.
	Dental Dam has a specified shelf life. Do not use the device after the Use-by Date because the device functionality and performance characteristics may be compromised.
	Caution: Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare practitioner
	Do not store near latex products.

## Additional Information

**Note:** Dispose of device in accordance with local, regional, national, and/or international regulations. Devices used intraorally should be treated as a biohazard and disposed of properly per local regulations.

**Note:** Upon request, a hardcopy of the IFU will be supplied free of charge.

**For users located in the EU:** Any serious incident that has occurred in relation to the use of the device should be reported to Coltène/Whaledent, Inc. and the competent authority of the member state in which the user and/or patient is established.

## Symbol Glossary

Symbol	Title and description of symbol
	Caution
 www.coltene.com	Consult electronic instructions for use - Indicates the need for the user to consult the instructions for use
	Medical Device – Indicates the item is a medical device
	Sale Restricted: The purchase of the device is restricted to licensed healthcare practitioners only.
	Use-by Date - Indicates the date after which the medical device is not to be used
	Catalog Number - Indicates the manufacturer's catalogue number so that the medical device can be identified
	Do not re-use - Indicates a medical device that is intended for use on a single patient during a single procedure
	CE mark

Symbol	Title and description of symbol
	Importer - Indicates the entity importing the medical device into the locale
	Manufacturer - Indicates the medical device manufacturer
	Batch code - Indicates the manufacturer's batch code so that the batch or lot can be identified
	Authorized representative in the European Community - Indicates the authorized representative in the European Community
	Not made with natural rubber latex

## Storage

Store Dental Dam between 50°F - 104°F (10°C - 40°C)..

Keep dry and keep away from heat.

Symbol	Title and description of symbol
	Keep dry - Indicates a medical device that needs to be protected from moisture
	Keep away from heat - Indicates a medical device that needs protection from heat
	Temperature Limit - Indicates the temperature limits to which the medical device can be safely exposed. The upper and lower limits of temperature shall be indicated adjacent to the upper and lower horizontal lines.

# HySolate SyntX Dam

Non-Latex Pre-Printed  
Polyisoprene Dental Dam

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Dent4You AG
Coltene/Whaledent Inc.
Coltene/Whaledent Ltd.
Coltene Whaledent GmbH + Co. KG