Dental Conservative – Restorative Instruments

Burnishers, Spatulas, Pluggers, Cord Packers, Base Carriers, Carvers
and Condensers

ErgoDenta ApS

Hørskættens 14, 2630 Taastrup, Denmark.

Tel: +4555241100

Email: info@ergodenta.dk

Web: www.ergodenta.dk

en-English

Attention

The operator/user must carefully read and understand this manual thoroughly to keep the product performance durable and reliable for defined life expectancy. After opening the packaging, first of all it is necessary to check the component against the standard configuration. Check that these are all present and in perfect condition.

Intended Use:

Our Conservative instruments are intended for the placement, shaping, and adaptation of composite and other restorative materials during dental restorative procedures. They assist in achieving proper contour, marginal integrity, and surface finish of restorations.

The Instrument Specific Intended Use is given below:

Burnishers:

Intended to be used for smoothing and contouring restorative materials to enhance marginal adaptation and surface finish.

Spatulas

Intended to be used for mixing, handling, and applying dental materials, such as composites, cements, and impression materials.

Pluggers

Intended to be used for compacting and condensing restorative materials into prepared cavities to ensure proper adaptation and density.

Cord Packer

Intended to be used for placing and packing retraction cord into the gingival sulcus to facilitate gingival retraction and improve impression accuracy.

Base Carrier

Intended to be used for transferring and applying base or liner materials, such as calcium hydroxide or glass ionomer, to protect the pulp and support restorations.

Carver

Intended to be used for shaping and contouring restorative materials to replicate natural tooth anatomy and refine occlusal surfaces.

 Our Dental Conservative Instruments are intended to be used by qualified dental professionals only in a professional dental facility.

Patient population:

These instruments are intended for use on adult patients and children requiring dental restorative procedures as determined by a qualified dental professional. The appropriate instrument type and size must be selected based on the patient's oral anatomy and the specific clinical requirements of the restorative treatment.

For Available Configurations/Models please visit the Restoration section in our Catalogue

Features

- Manufactured from high-grade, autoclavable stainless steel, ensuring durability, corrosion resistance, and long-term sharpness retention.
- Finely contoured tips and ergonomic handles provide precise manipulation of dental materials, enhancing control and reducing hand fatigue during procedures.
- Variety of tip shapes and sizes (rounded, flat, pointed, or serrated) designed for diverse clinical applications, including carving, packing, condensing, smoothing, and contouring restorative materials.
- Balanced shank-to-blade ratio ensures controlled flexibility while maintaining structural integrity, optimizing efficiency in restorative techniques.
- Handles available in textured or silicone-molded surfaces to provide a secure, comfortable grip, improving precision and stability.

Operating instructions

 These instruments are non-sterile and must be cleaned and sterilized before first use.

 Always check if the reprocessed instruments are in perfect condition and do not use the instruments if any kind of deterioration, bent or breakage is observed.

 Use a firm but controlled grip to maintain precision and stability. Avoid excessive force that may compromise the integrity of the instrument or restorative material.

For Burnishers:

- Apply light pressure in circular motions to smooth and adapt restorative materials.
- Use after placement to refine contours and achieve a polished finish.

For Spatulas:

- Mix dental materials thoroughly before application.
- Ensure consistency and accurate placement when transferring materials.

For Pluggers & Condensers:

- Compact restorative materials in small increments to avoid air pockets.
- Use firm, vertical pressure for uniform adaptation.

For Cord Packers:

- Gently insert the gingival retraction cord without damaging soft tissues.
- Ensure proper cord placement for optimal tissue retraction and moisture control.

For Base Carriers:

- Load base material securely before transfer to the prepared cavity.
- Place material precisely to ensure even distribution.

For Carvers:

- Shape and contour restorative material to match natural tooth anatomy.
- Remove excess material carefully to prevent over-carving.

For Cord Packers

- Gently insert the gingival retraction cord without damaging soft tissues.
- Ensure proper cord placement for optimal tissue retraction and moisture control.

Adverse Event

Any serious incident that has occurred concerning the device should be reported to the manufacturer and to the respective competent authority of the Member State in which the incident has occurred.

Cleaning & Sterilization

Cleaning

 Clean the device with care after each application to avoid the drying of contaminants. Residues of restorative materials must be removed immediately.

Initial Cleaning

1. Remove any visible debris from the instrument using moisten a cotton swab or cloth with isopropyl or ethyl alcohol and gently wiping the surface of the product.
2. Use a soft brush for cleaning the instrument if required.

Manual Cleaning

3. Prepare a solution of neutral pH enzymatic detergent according to the manufacturer's instructions.
4. Immerse in a pre-soak enzymatic cleaner solution.
5. Thoroughly clean all surfaces of the Instrument using a soft brush or cloth. Pay close attention to any crevices or hard-to-reach areas.
6. Rinse thoroughly under lukewarm running tap water for a minimum of 30 seconds to remove all traces of detergent.

Automated Cleaning

1. Always clean in an appropriate box or cassette and ensure that they are properly positioned within the unit.
2. Always follow the equipment manufacturer's instructions and ensure the instruments are compatible with the cleaning system.

3. Visually inspect for cleanliness, If any contamination is visible, repeat the cleaning steps until required cleanliness is achieved.
4. Dry with clean, lint free cloth or filtered compressed air until there is no visible moisture. Follow STEAM STERILIZATION PROCEDURE.

 Ultrasonic cleaning may be performed in accordance with hospital or facility protocols. Ensure proper rinsing and drying after the process to prevent residue buildup.

Sterilization

Note: Ensure that the instruments are completely dry before sterilization.

Steam Autoclaving

1. Place the instrument in an approved sterilization pouch or wrap.
2. Sterilize using a steam autoclave following a validated sterilization cycle according to ISO 17665-1 and your facility's protocols. A typical recommended cycle is 132-134°C (270-273°F) for 4 minutes.
3. Allow the instruments to cool completely before handling. Store in a clean, dry, and controlled environment until ready for use.
4. Follow the pouch/wrap manufacturer instructions for storage conditions and maximum storage time.

Important Notes

- Always Inspect the instruments for any signs of deterioration, breakage and bent etc. after each sterilization cycle. Discard if damage is observed.
- Always follow your facility's infection control protocols.
- Refer to your autoclave manufacturer's instructions for proper operation and maintenance. The Sterilization equipment must be validated by the hospital and or sterilization equipment manufacturers.
- The above-mentioned cleaning and sterilization guidelines, provided by manufacturer, are intended as procedures compatible with specific materials. Sterilization must be performed according to the Hospital/Clinic approved protocol.
- The responsibility for ensuring proper sterilization of instruments lies with the user. The effectiveness of sterilization depends on validated cleaning, packaging, and sterilization procedures carried out at the facility.

Disposal

Our conservative instruments: Burnishers, Spatulas, Pluggers, Cord Packers, Base Carriers, Carvers, and Condensers contain no hazardous materials. However, these must be cleaned/sterilized as per defined procedures before disposition. Please follow the local and national regulations or healthcare facilities' defined disposal and waste management policies.

Disclaimer

The product must be used, reprocessed, and maintained strictly in accordance with the instructions provided above. Any deviation from these guidelines by the dental professional or user is undertaken at the user's sole risk. The Manufacturer will not accept any requests for refunds or exchanges under warranty for products that have not been handled and reprocessed in compliance with these instructions.

Explanation of utilized symbols

	Medical Device Compliant with Regulation (EU) 2017/745
	Consult instructions for use
	Medical Device
	Product Code/Catalogue Number
	Model/Variant
	Lot Number
	The Device is supplied non-sterile.
	Sterilizable in a steam sterilizer (autoclave) at 132-134°C (270-273°F)
	Manufacturing Date
	Manufacturer
	Keep in a cool, dry place
	Caution, read instructions

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