GREEN&CLEAN SK

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SMQS by Arnold Bott AG, Industriestrasse 59, 8152 Glattbrugg / CH | www.smqs.ch

LOT

EN - Intended use: GREEN&CLEAN SK is a non-alcoholic for disinfection of surfaces of non-invasive medical devices and for all other areas where a proper hygienic condition is required. This is used for disinfection of alcohol sensitive surfaces like leather or synthetic leather covers. **Indication:** Not applicable. **Contraindication:** Not for using for Stainless steel, which is prone to chloride corrosion. **Target group:** Professionally trained health workers. **General safety-related information:** All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident. **Danger and safety information:** The mixture is classified as non-hazardous according to (EC) regulation no. 1272/2008 (CLP). **Risks and side effects:** This product does not have direct skin contact with the user or the patient. **Product description:** GREEN&CLEAN SK is a ready-to-use solution and can be used as a spray or wipe disinfectant for alcohol-sensitive surfaces. **Application of spray disinfection:** Spray the undiluted solution directly onto the surface to be disinfected and keep it moist for the duration of the exposure time. **Application wipe disinfection:** Put the undiluted solution on a wipe. Then completely wet the surface to be disinfected and keep it moist for the duration of the exposure time. **Composition:** Spray and wipe disinfection based on quaternary ammonium compounds.

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METASYS Medizintechnik GmbH, Florianistraße 3, 6063 Rum, Austria | Tel.: +43 512 205420 | www.metasys.com