# LuxaForm

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English

### Product description

LuxaForm is a thermoplastic molding polymer.

### Intended purpose

Impressions for provisional repairs.

### Limitations of use

Do not use LuxaForm to make an impression of more than 5 teeth.

### Indications

All dental situations requiring indirect restorations e.g. tooth loss, caries and trauma.

### Contraindications

Do not use the material where there are known allergies to any of the components or contact allergies.

### Patient target group

Persons receiving treatment as part of a dental procedure.

### Intended users

Dentist, dental assistant staff

### Notes for use

- Softened LuxaForm sticks to gloves. Always knead LuxaForm without gloves. For insertion in the mouth, isolate the fingertips of the gloves with Vaseline.
- In order to ensure a safe removal, do not mold LuxaForm too tightly around the teeth when making an impression in the mouth.
- Block out any undercuts with an appropriate agent, e.g. wax.
- Do not use LuxaForm to make an impression of more than 5 teeth.
- Perform any necessary post-processing of the impression with a scalpel.
- The impressions retain their form for months and can also be re-used later, e.g. if a temporary restoration breaks.

## Recommended use

1. Place one LuxaForm tablet per tooth imprint in approx. 70 °C hot water, until the material becomes soft and clear (approx. 01:00 minute). For multiple imprints, use more tablets as needed. The tablets can be kneaded together once they have become transparent.
2. Shape the transparent material into a cube and press it around the teeth from which the impression is to be taken.

**Note:** After cooling, the material quickly becomes hard and turns opaque blue. It can be cooled more rapidly by briefly spraying with water.

3. Remove the hardened LuxaForm from the tooth and lightly isolate it with Vaseline or another separating agent.

**Note:** When using temporary crown and bridge materials, their initial temperature should not exceed 25 °C.

4. Apply temporary crown and bridge material (e.g. Luxatemp Star) in the usual fashion.

## Interactions

Do not use with MMA-containing provisional crown and bridge materials.

## Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out.

## Warnings/precautions

- For dental use only!
- Keep out of the reach of children!
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

## Performance characteristics of the device

Stable up to approx. 55 °C.

## Storage/disposal

- Store in a dry place at 15 to 25°C (59 to 77°F)!
- Protect against exposure to direct sunlight!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

## Composition

Polycaprolactone (approx. 98 %), additives, pigments

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