## X Ultra-Blend® plus Cavity Liner and Pulp-Capping Material

Ultra-Blend plus liner is a unique dentin-adhering liner and base material containing calcium hydroxide and calcium hydroxyapatite in a urethane dimethacrylate (UDMA) base. It's perfect for pulp capping and will not dissolve over time. Made of a light-activated, radiopaque material, Ultra-Blend plus liner is highly filled for minimal shrinkage.

- Bioactive<sup>1</sup>
- Superior calcium release<sup>2</sup>
- Controlled, precise syringe delivery
- No mixing necessary
- Light cured when clinician is ready
- Radiopaque



Jirra-Blend® P Dentin Liner & Prolective

800.552.5512 ULTRADENT.COM

## 🔇 Ultra-Blend®*plus*

## Ultra-Blend® plus liner application:


A bioactive base material is an essential tool used in restorative dentistry—especially in cases where the pulp is exposed. Thanks to its superior calcium release, Ultra-Blend plus liner effectively stimulates the reparative process.<sup>1</sup> That's one reason it's been a preferred pulp-capping material for over 15 years.



Use Ultra-Blend plus liner near pulp (pink) and for small nonhyperemic exposure. If larger exposure and/or hyperemic pulp, initiate endodontic therapy.




Apply Consepsis<sup>®</sup> antibacterial solution with the plastic Blue Mini<sup>®</sup> Dento-Infusor<sup>®</sup> tip for 60 seconds with gentle scrubbing action. Gently air dry (don't rinse).

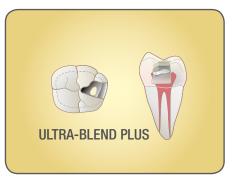

With the Black Micro® tip, apply Ultra-Blend plus liner to dry dentin at exposure border (or over thin pink dentin); cover exposure and light cure. Minimize dentin coverage to maximize available dentin for bonding.



Apply Ultra-Etch® 35% phosphoric acid etchant solution for 20 seconds. Suction, rinse, and dry until damp.

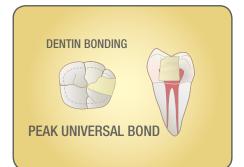

With the Inspiral<sup>®</sup> Brush tip, apply Peak<sup>®</sup> Universal Bond, rubbing and agitating into the preparation for 10 seconds. Air thin at half pressure for 2–3 seconds and light cure for 10 seconds with the VALO<sup>®</sup> curing light on Standard mode. Restore with a quality composite.

## 415 Ultra-Blend plus Kit

2 x 1.2 ml Ultra-Blend plus Dentin syringes 2 x 1.2 ml Ultra-Blend plus Opaque White syringes 20 x Black Micro tips 20 x Black Mini tips



416 Ultra-Blend plus Dentin Refill

417 Ultra-Blend plus Opaque White Refill

4 x 1.2 ml syringes





800.552.5512 ULTRADENT.COM

© 2017 Ultradent Products, Inc. All Rights Reserved. 15706.6 112116

Improving Oral Health Globally