<b>ESPE</b>
<b>Ketac™ Fil Plus Aplicap™</b>
<b><span>en</span> Glass Ionomer Filling Material</b>
<b><span>de</span> Glasionomer Füllungsmaterial</b>
<b><span>fr</span> Matériau d’obturation vitro-ionomère</b>
<b><span>it</span> Materiale di riempimento a base di vetroionomero</b>
<b><span>es</span> Material de relleno de ionómero vítreo</b>
<b><span>pt</span> Material de obturação de ionómero vítreo</b>
<b><span>nl</span> Glasionomeer vulmateriaal</b>
<b><span>el</span> Υαλοίονομερες εμφρακτικό υλικό</b>
<b><span>sv</span> Glasionomer-fyllningsmaterial</b>
<b><span>fi</span> Lasi-ionomeeri-täyttemateriaali</b>
<b><span>da</span> Glasionomer-fyldningsmateriale</b>
<b><span>no</span> Glassionomer fyllingsmateriale</b>

Instructions for Use	Gebruiksinformatie
Gebrauchsinformation	Πληροφορίες χρήσεως
Mode d'emploi	Bruksanvisning
Informazioni per l'uso	Käyttöinfo
Información de uso	Brugsanvisning
Instruções de uso	Bruksinformasjon

<span><span></span></span> 3M Deutschland GmbH	<span><span></span></span> 0123
Dental Products <div>Carl-Schurz-Str. 1</div> <div>41453 Neuss – Germany</div>	<span><span></span></span>
<div> <div> <div>3M ESPE Customer Care/MSDS Information: U.S.A. 1-800-634-2249 and Canada 1-888-363-3685.</div> <div> <p>3M, ESPE, Aplicap, CapMix, Ketac and RotoMix are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 2014, 3M. All rights reserved.</p> </div> </div> </div>	<span><span></span></span>
<p><span><span></span></span></p>	<span><span></span></span>

<b><span>en</span> ENGLISH</b>
--------------------------------

**Product Description**
Ketac™ Fil Plus Aplicap™ is a glass ionomer filling material in capsules. The following colors are available: A1/bright, A2/bright-yellow, A3/yellow, A3.5/brown, A4/yellow-brown, C2/gray, C3/gray-brown, DGO/dark-gray. Minimally 0,14 ml of the material are dispensable from each capsule. For details on all products mentioned below please refer to the corresponding Instructions for Use.

☞ Instructions for Use should not be discarded for the duration of product use.

**Areas of Application**

- Class III and V fillings
- Wedge-shaped defects
- Class I small fillings
- Primary tooth fillings
- Restorative fillings prior to crown preparation
- Fissure sealing

**Precautionary measures**

3M MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

**Preparation**

- Remove the carious substance only, no undercuts required.
- Do not prepare thin edges since the material requires a wall thickness of 0,5 mm.

**Pulp protection**

► In order to prevent irritations of the dental pulp, areas close to the pulp should be covered punctiformly with a calcium hydroxide preparation.

**Conditioning**

- For optimal chemical bonding to the teeth the smear layer produced in the preparatory steps must be carefully removed by applying Ketac Conditioner to the prepared surfaces and allowing the substance to react for 10 sec.
- Then rinse with copious amounts of water.
- Blow the cavity dry in only 2–3 short intervals with water and oil-free air or dab dry with cotton pellets. Do not over-dry! **The cavity should be just so dry that the surface has a matt shiny appearance.** Excessive drying can result in post-operative sensitivity after filling.
- Avoid further contamination.

**Activation**

- Place the Aplicap™ Activator on a sturdy surface and insert the Aplicap capsule in the activator.
- Using the ball of your hands, depress the activator lever firmly and completely to the stop and hold it down for 2 to 4 seconds.
  - Firmly and completely depressing the lever to the stop and holding it in this position is the only way to ensure that all of the liquid is pressed into the powder. If inadequate pressure is applied or if the lever is not depressed completely, it is possible that too little litle will be mixed with the powder, increasing the viscosity and changing the product characteristics.

**Mixing**

► Mix the capsules at approx. 4,300 rpm, in a high frequency mixing device such as CapMix™ (for 10 sec) or the RotoMix™ rotating mixer (for 8 sec),

**Application**

- Apply with the Aplicap™ Applier. Both the Aplicap Activator and the Aplicap Applier are color-coded in red.
- Ensure that there is no contamination from water, saliva, etc., entering the working area for the entire duration of application; the working area must be kept dry.

Attention

- Avoid any lag times between the processes of activation, mixing, and application, as the material is in the process of setting which may impair or prevent application of the material.
- Ketac Fil Plus Aplicap sticks to metal instruments and should be washed off with cold water prior to setting.

**Times**

At an ambient temperature of 23° C/73° F the following times apply:

	min:sec
Activation of capsule	0:02
Mixing with RotoMix	0:08
or	
Mixing with high frequency mixing device such as CapMix	0:10
Processing from start of mixing	1:30
Setting from start of mixing	7:00

At ambient temperatures above 23° C/73° F the setting process is accelerated. Exceeding the indicated processing time may lead to reduced bonding to enamel and dentin.

Shorter mixing times prolong the processing and setting times, whereas longer mixing times shorten the processing and setting times. The mixing time may be reduced to 8–10 sec where high frequency mixing devices are used, and to 6–8 sec with the RotoMix device.

**Filling protection**

In the first phase of their 2-phase setting process, glass ionomers are sensitive to moisture. Subsequently, there is the possibility of dehydration which may be prevented by the use of Ketac™ Glaze directly after modeling.

- Apply Ketac Glaze to all exposed surfaces of the glass ionomer cement and cure with light for 10 sec.

**Finishing**

► Beginning no earlier than 7 minutes after the start of mixing, use Arkansas stone, fine-grain diamonds, polishing discs of decreasing grain size or silicon polishers for finishing.

► If desired, apply Ketac Glaze again for surface sealing, and cure with light for 10 sec.

**Storage and Shelf-Life**

Store the product at 15–25° C/59–77° F. Do not use after the expiry date.

**Customer Information**

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

**Warranty**

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the

product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH’s sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

**Limitation of Liability**

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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