# roeko Retracto

NON-IMPREGNATED

## Instructions for use

**Use:**

A product for improving the results of dental impressions.

**Application:**

Cut the *Retracto* to the required length. *Retracto* can be soaked in GINGIVA LIQUID or another gingival retraction solution of your choice.

**Please note:** Don't use GINGIVA LIQUID in cases of necrotizing Gingivitis ulcerosa. Please follow instructions for use of the chosen liquid. GINGIVA LIQUID is not available in all countries.

Then apply the cord tightly to the cervix of the tooth and press carefully into the sulcus by means of a retraction cord packer.

Leave the non impregnated cord for at least 5 minutes in the sulcus. When using GINGIVA LIQUID time of effectiveness is 1-3 minutes. Remove the cord directly before taking the impression. The non impregnated retraction cord is available in 2 versions: *Retracto* non impregnated braided is slightly more compact and firmer than *Retracto* non impregnated twisted and is preferable when increased retraction is required. Both versions are available in x-fine, fine, medium and thick.

<b>Braided</b> non impregnated		
0	x-fine	REF 511 000
1	fine	REF 511 001
2	medium	REF 511 002
3	thick	REF 511 003

<b>Twisted</b> non impregnated		
0	x-fine	REF 523 000
1	fine	REF 523 001
2	medium	REF 523 002
3	thick	REF 523 003

**Contraindications:**

None

**Warning:**

Medical devices should be kept out of reach of children.

**Use and storage:**

- for single use only
- for dental use only
- the pack has to be closed immediately after use and to be stored in a dry place
- shelf life 5 years
- this medical device should not be used after the expiration date
- unpacked thread can be disposed of as „biological waste“. The plastic container can be recycled.

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